tr

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 5123-1220
1063833 Rev. 02

en

***digene*[®] Female Swab Specimen Collection Kit Contains:**

1 x 2  specimen collection swabs   

1 x 1 ml  Specimen Transport Medium (contains 0.05% sodium azide)

The CE marking indicates that this product meets requirements of the European Directive 93/42/EEC relating to medical devices.

Precautions and Warnings

Use only with *digene* HC2 CT/GC DNA Tests. Specimens collected with this system cannot be used for culture.

Caution:

 **For the United States Only: Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner.**

- **For professional use only. Read these instructions carefully before taking the cervical specimen.**
- **For the collection and transport of cervical specimens is for single use only; reuse may cause a risk of infection and/or inaccurate results.**
- **Some kit components contain sodium azide, which may react with lead or copper plumbing to form highly explosive metal azide compounds. Dispose of in accordance with local regulations. Contact your local QIAGEN Representative to obtain Material Safety Data Sheets.**
- **Specimen Transport Medium: Warning! Causes mild skin irritation. If skin irritation occurs: Get medical advice/ attention.**

Specimen Transport Medium

Dispose of in accordance with local regulations.

Storage Instructions

Store kits at 15-30°C. Do not use after the expiration date indicated next to the symbol  on pouch label.

Specimen Collection and Handling

Note: Collect Pap smear specimen before obtaining specimen for DNA testing. Collect DNA specimen prior to application of acetic acid or iodine if a colposcopy will be performed.

Remove excess mucus from the cervical os and surrounding ectocervix using 1 of the 2 specimen collection swabs provided. Discard the swab in accordance with local regulations.

Insert the second swab into the endocervical canal, and rotate it through 5-180° turns, alternating in opposite directions.

Rub swab firmly over the entire area of the transformation zone.

Withdraw the swab carefully; avoid contact with vaginal mucosa.

Insert the swab to the bottom of the transport tube. Snap off the shaft at the score line, and then cap the tube securely.

Specimen Transport

Warning! Causes mild skin irritation. If skin irritation occurs: Get medical advice/ attention.

Cervical specimens may be held for up to 2 weeks at room temperature and shipped without refrigeration to the testing site. At the testing laboratory, specimens can be stored an additional week at 2–8°C or 3 months at -20°C prior to testing. Refer to the *digene* HC2 CT/GC DNA Test system package inserts for instructions on how to run the assays. To prevent caps from popping off specimen tubes that are shipped or stored frozen, cover caps with Parafilm[®] prior to shipment.

For domestic and international shipments, specimens should be packaged and labeled in compliance to applicable State, Federal, and international regulations covering the transport of clinical specimens and infectious substances.

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©2015 QIAGEN all rights reserve.