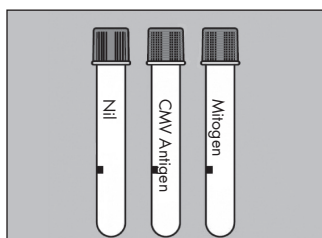
QuantiFERON®-CMV Blood Collection Tubes Single Patient Pack

Cat.No: 0192-0301

CONTENTS

3x 1 ml QF-CMV blood collection tubes (Nil, CMV Antigen, Mitogen)
1x Package Insert


For more information, please refer to the *QuantiFERON-CMV Package Insert*, which can be found at www.QuantiFERON.com.

For use with QuantiFERON-CMV ELISA kit (Cat. no. 0350-0201).

1

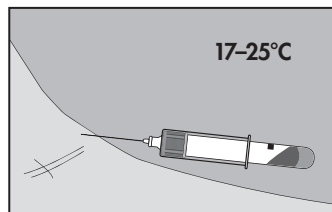BLOOD COLLECTION

Collect blood by venepuncture.
*Vacuum fills to black mark (1 ml).
Blood flow may be slow.
Hold tube on needle for 2–3 seconds after flow ceases.
Repeat tube if not close to mark.*

Technical Tip

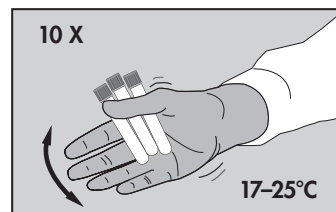
Butterfly needles—prime the tubing with a purge tube (not supplied) before filling QuantiFERON-CMV tubes.

WARNING:

- Standard blood handling precautions apply.
- Tubes may be stored at 4°C–25°C and must be at room temperature (17°C–25°C) at the time of use.

2

BLOOD COLLECTION

Immediately after filling tubes, shake them ten (10) times, just firmly enough to ensure the entire inner surface of the tube is coated with blood, to dissolve antigens on tube walls.

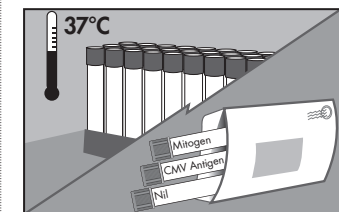
*Tubes should be between 17–25°C at the time of blood filling.
Over-energetic shaking may cause gel disruption and could lead to aberrant results.*

Label tubes appropriately.

3

SHIPPING / INCUBATION

OPTION 1
Incubate at Laboratory

Ship tubes to laboratory at 17–27°C. Blood must be incubated at 37°C as soon as possible (and within 16 hours of collection).

Laboratory staff must remix the tubes immediately prior to 37°C incubation.

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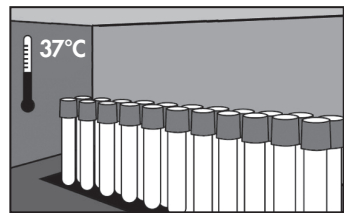
SHIPPING / INCUBATION

OPTION 2
Incubate at
Collection site

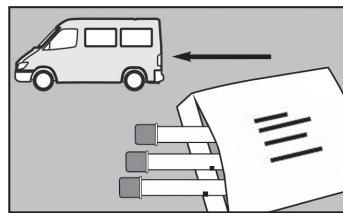
5

Blood must be incubated as soon as possible (and within 16 hours of collection). Incubate tubes upright at 37°C for 16–24 hours.

Incubators may be available from QIAGEN.

Technical Tip
Humidity/CO₂ not required.

If tubes are not incubated at 37°C soon after collection, re-mixing of the tubes by inverting 10 times must be performed immediately prior to incubation.


SHIPPING / INCUBATION

OPTION 2
(continued)

6

Ship incubated tubes to testing laboratory (within 3 days, if not centrifuged).
Maintain tubes at 4–27°C.

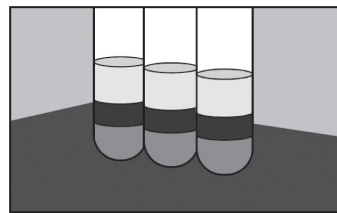Technical Tip
Label tubes as “Incubated”.

CENTRIFUGATION

7

Centrifuge tubes prior to sampling.
Spin at 2000–3000g for 15 minutes.

Plasma in centrifuged tubes is stable for 28 days when stored at 2–8°C.

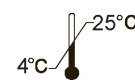
For up-to-date licensing information and product-specific disclaimers, see the QuantiFERON-CMV ELISA Package Insert.

www.QuantiFERON.com

CONTACT DETAILS

QIAGEN
Asia-Pacific: techservice-ap@qiagen.com
Europe: techserviceQFT-eu@qiagen.com
Middle East/Africa: techserviceQFT-eu@qiagen.com
USA/Canada: techservice-na@qiagen.com
Latin America (not including Brazil or Mexico): techservice-latam@qiagen.com
Mexico: techservice-MX@qiagen.com
Brazil: techsebr@qiagen.com
www.QuantiFERON.com

QIAGEN
19300 Germantown Road
Germantown, MD 20874
USA
+1-800-426-8157

QIAGEN GmbH
QIAGEN Strasse 1
40724 Hilden
Germany

Trademarks: QIAGEN®, QuantiFERON® (QIAGEN Group).
© 2015 QIAGEN, all rights reserved.
1075109 Rev. 03 08/2015

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